SoloSTAR[®] Quick Reference Guide

For patients using the Apidra[®] and/or the Lantus[®] SoloSTAR[®]


These instructions are supplied as a guide only. Always read the full instruction leaflet before you use the SoloSTAR[®] ready-to-use injection pen for the first time. If you have any questions, ask your healthcare professional.

1 Check the insulin

- Verify you have the right insulin.
- Take off the pen cap and check the appearance of the insulin, it should be clear.


2 Attach the needle

- Wipe the rubber seal with alcohol.
- Remove the protective seal, and attach a new needle.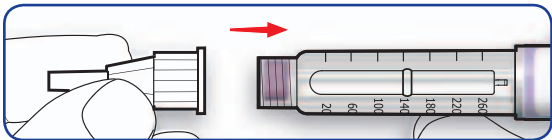


3 Do a safety test

- Set a dose of 2 units.

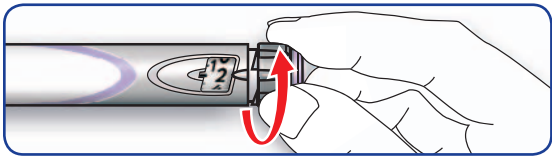
- Take off both needle caps.
- Hold the pen with the needle pointing up.
- Tap the insulin reservoir so that any air bubbles rise up towards the needle.
- Press the injection button all the way in and check that insulin comes out.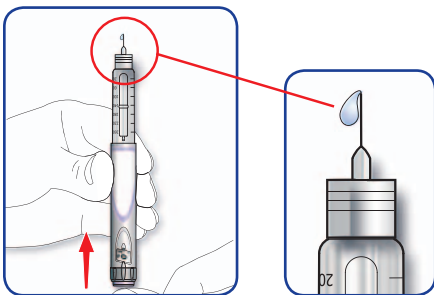

- If insulin does not come out, repeat the test.
- If no insulin comes out after 3 attempts, attach a new needle and try again.


4 Set your dose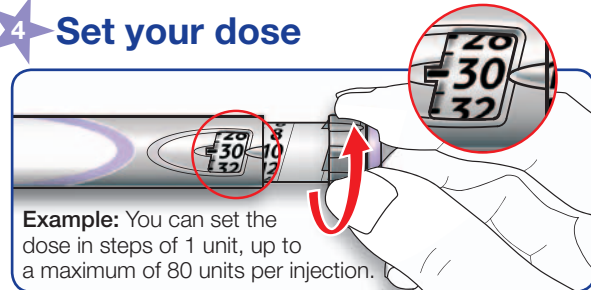

Example: You can set the dose in steps of 1 unit, up to a maximum of 80 units per injection.

5 Inject your dose

- Insert the needle into the skin, as instructed by your healthcare professional.
- Press the button all the way in.
- Keep the button pressed in and slowly count to 10.
- Withdraw the needle.

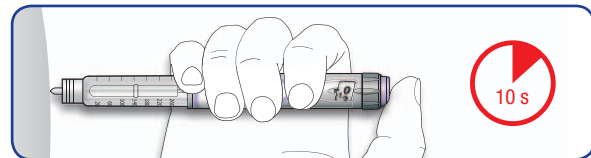6 Finish

- Put the outer needle cap back on and use it to unscrew the needle from the pen.
- Dispose of the needle safely, e.g., in a sharps container.
- Put the pen cap back on.

24 24-HOUR SUPPORT
1.888.852.6887

! IF YOU CAN'T dial to the dose you want, check if you have enough insulin in the reservoir.
IF NO INSULIN COMES OUT after changing the needle, your SoloSTAR[®] may be damaged. Do not use this SoloSTAR[®].
EACH SoloSTAR[®] is for use by one person only.

For more information about your medication or your injection pen, please visit our website at www.sanofi.ca.

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